


ASAP[®]

INDIRECT+

ALL SURFACE ACCESS POLISHERS

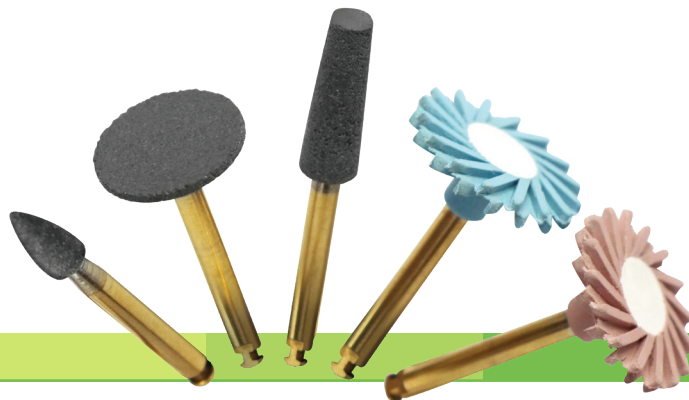

FINISHING & POLISHING

The **ASAP INDIRECT+** diamond polishing system is all you need to refine and create an outstanding shine on all your indirect ceramic restorations. This versatile system is fast and simple; delivering a super high-gloss polish A.S.A.P.! Designed with a higher concentration of diamond particles and coupled with a durable silicone binder, the **ASAP INDIRECT+** system is ideal for today's harder indirect restorative surfaces.





Available in an intraoral (latch) option, the **ASAP INDIRECT+** Starter Kit includes three multi-shaped, diamond impregnated "adjusters", as well as two **ASAP INDIRECT+** diamond impregnated polishing wheels. To make minor adjustments or to remove scratches caused by diamond bur adjustments, the **ASAP INDIRECT+** Coarse Adjuster (grey); available in a disc, point or cylinder shape, can be used on any zirconia, lithium disilicate or ceramic prosthesis. Next, use the **ASAP INDIRECT+** Pre-Polisher (blue) to smooth the prosthetic surface and initiate an initial gloss. The final step of this technique, is to apply light pressure using the **ASAP INDIRECT+** Final Polisher (pink) to create a stunning high luster polished surface, in seconds!

ASAP INDIRECT+ All Surface Access Polishers can access any surface: occlusal, lingual, interproximal and facial, eliminating the need for multiple shapes.

CLINICAL TECHNIQUE

Chair-side Adjustment and Polishing with the ASAP INDIRECT+ System

Dentistry and photography courtesy of Dr. Robert Lowe

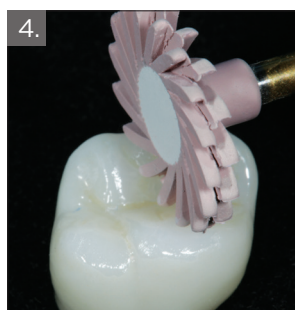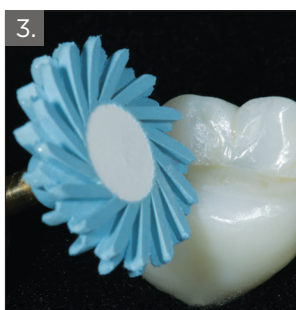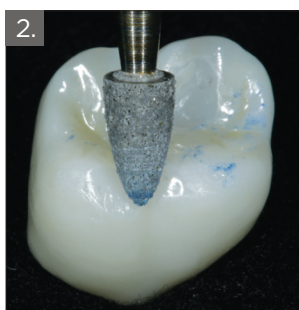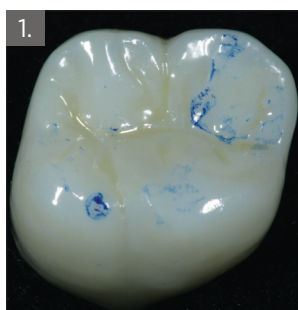

1. To make minor adjustments or to remove scratches caused by diamond bur adjustments, the **ASAP INDIRECT+** Coarse Adjuster; available in a disc, point or cylinder shape, can be used on any zirconia, lithium disilicate or ceramic prosthesis.

2. The **ASAP INDIRECT+** Pre-Polisher is used to smooth the prosthetic surface and initiate an initial gloss.

3. The **ASAP INDIRECT+** Final Polisher creates a stunning high luster in seconds!

4. **ASAP INDIRECT+** restores a high luster polish on any zirconia, lithium disilicate or ceramic surface, in under a minute!

CLINICIAN'S CHOICE[®]
DENTAL PRODUCTS INC.