

# A.S.A.P.<sup>®</sup>

## All Surface Access Polishers

### One Shape. All Access. Unlimited Shine.

The ideal polish provides life-like vitality to your esthetic restorations. It also promotes restorative longevity, prevents staining and improves tissue health along the margins. The challenge has always been how to achieve a great polish, quickly – until now!

**A.S.A.P. All Surface Access Polishers** from **CLINICIAN'S CHOICE<sup>®</sup>** provide the ultimate high shine, in less than a minute – with only one shape. These diamond-impregnated, flexible polishing spirals are designed for optimal adaptation to any surface, including difficult to reach areas, such as interproximal surfaces, and are ideal for use on most ceramics and zirconia, lithium disilicate, provisional materials and resin modified glass ionomers.

### A.S.A.P. Pre-Polisher

Embedded with diamond particles (44 microns), the **A.S.A.P. Pre-Polisher** (purple) gently diminishes small surface defects, without affecting anatomy, and prepares the surface for a final high gloss polish.

### A.S.A.P. Final High Shine Polisher

The second step to this time-saving technique is to apply light to moderate pressure in a circular motion over the restoration to create a smooth, final high luster surface. Each **A.S.A.P. Final High Shine Polisher** (orange) is embedded with small, 3-6 micron sized diamond particles to provide a life-like shine in as little as 20 seconds.

Autoclavable 30 times or more, **A.S.A.P. Polishers** are designed to withstand wear while providing an efficient and consistent polish use after use. **A.S.A.P. Polishers** are available in both a small (10mm) for pedodontic restorations and patients with a restricted bite, as well as a large (14mm) polisher, in a variety of kit and refill configurations.



A.S.A.P. Pre-Polisher



A.S.A.P. Final High Shine Polisher



Buccal



Lingual




Interproximal



Labial



Occlusal

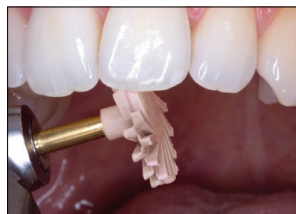

Incisal

#### A.S.A.P. STARTER KIT

Contains: 3 x Pre-polishers Small, 3 x Pre-polishers Large, 3 x Final High Shine Polishers Small, 3 x Final High Shine Polishers Large

#### A.S.A.P. 6-PACK REFILL SMALL

Contains: 3 x Pre-polishers, 3 x Final High Shine Polishers

#### A.S.A.P. 6-PACK REFILL LARGE

Contains: 3 x Pre-polishers, 3 x Final High Shine Polishers

#### A.S.A.P. 2-PACK REFILL SMALL

Contains: 1 x Pre-polisher, 1 x Final High Shine Polisher

#### A.S.A.P. 2-PACK REFILL LARGE

Contains: 1 x Pre-polisher, 1 x Final High Shine Polisher

#### A.S.A.P. 3-PACK PRE-POLISHER SMALL

#### A.S.A.P. 3-PACK PRE-POLISHER LARGE

#### A.S.A.P. 3-PACK FINAL HIGH SHINE POLISHER SMALL

#### A.S.A.P. 3-PACK FINAL HIGH SHINE POLISHER LARGE