



91% overall rating

ASAP[®] Indirect+ All Surface Access Polishers



The **ASAP[®] Indirect+** diamond polishing system is all you need to refine and create an outstanding shine on all your indirect ceramic restorations. This versatile system is fast and simple; delivering a super high-gloss polish A.S.A.P.! Designed with a higher concentration of diamond particles and coupled with a durable silicone binder, the **ASAP Indirect+** system is ideal for today's harder indirect restorative surfaces.





Available in Intraoral/RA or Extraoral/HP options, the **ASAP Indirect+** Starter Kits include multi-shaped, diamond impregnated "adjusters", as well as **ASAP Indirect+** diamond impregnated polishing wheels. To make minor adjustments or to remove scratches caused by diamond bur adjustments, the **ASAP Indirect+** Coarse Adjuster (grey); available in a disc, point or cylinder shape, can be used on any zirconia, lithium disilicate or ceramic prosthesis. Next, use the **ASAP Indirect+** Pre-Polisher (blue) to smooth the prosthetic surface and initiate an initial gloss. The final step of this technique, is to apply light pressure using the **ASAP Indirect+** Final Polisher (pink) to create a stunning high luster polished surface, in seconds!

ASAP Indirect+ All Surface Access Polishers can access any surface: occlusal, lingual, interproximal and facial, eliminating the need for multiple shapes.

CLINICAL TECHNIQUE

Dentistry and photography courtesy of Dr. Robert Lowe.

Chair-Side Adjustment and Polishing with the ASAP Indirect+ System

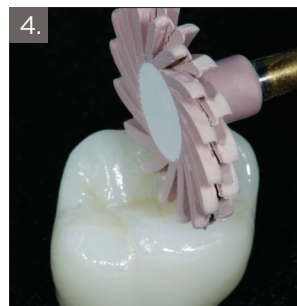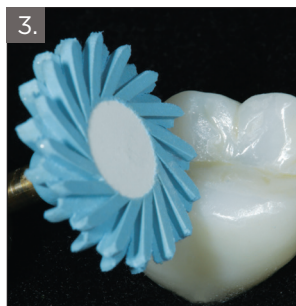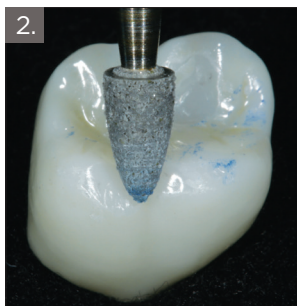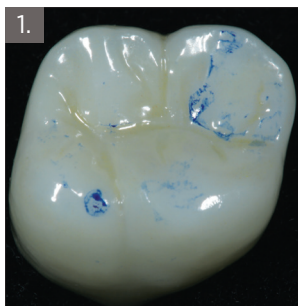

1. To make minor adjustments or to remove scratches caused by diamond bur adjustments, the ASAP Indirect+ Coarse Adjuster; available in a disc, point or cylinder shape, can be used on any zirconia, lithium disilicate or ceramic prosthesis.

2. The ASAP Indirect+ Pre-Polisher is used to smooth the prosthetic surface and initiate an initial gloss.

3. The ASAP Indirect+ Final Polisher creates a stunning high luster in seconds!

4. ASAP Indirect+ restores a high luster polish on any zirconia, lithium disilicate or ceramic surface, in under a minute!

<p>INTRAORAL/RA STARTER KIT Contains: 3 Coarse Adjusters (1 x Point, 1 x Disc, 1 x Cylinder), 1 x Pre-Polisher, 1 x Final Polisher</p>	<p>EXTRAORAL/HP STARTER KIT Contains: 2 Coarse Adjusters (1 x Disc, 1 x Cylinder), 2 x Pre-Polisher, 2 x Final Polisher</p>
<p>INTRAORAL/RA POLISHER REFILL 2-PACK Contains: 1 x Pre-Polisher, 1 x Final Polisher</p>	<p>EXTRAORAL/HP POLISHER REFILL 2-PACK Contains: 1 x Pre-Polisher, 1 x Final Polisher</p>
<p>INTRAORAL/RA REFILL 3-PACKS Pre-Polisher Final Polisher Point Adjuster Cylinder Adjuster Disc Adjuster</p>	<p>EXTRAORAL/HP REFILL 3-PACKS Pre-Polisher Final Polisher Cylinder Adjuster Disc Adjuster</p>