

PowerMix™

Automatic Impression Material Dispenser

Dispensing Power At Your Fingertips

Portable, fast and easy, the **PowerMix™ Automatic VPS Dispenser** from **CLINICIAN'S CHOICE**, delivers perfectly mixed, void-free impression material with the press of a button. With **PowerMix**, now you can fill a full arch impression tray in just 10-15 seconds – no more hand pumping. The **PowerMix** design has a small footprint stand for storage or direct dispensing and can easily be transported from operator to operator.

Fast & Convenient: Fill a full arch impression tray in 10-15 seconds with no voids or hand fatigue.

Lightweight & Portable: The PowerMix (2lbs) is easily transported between operatories for handheld or stand-delivered tray filling.

Long Battery Life: Fully charged, PowerMix dispenses 8-10 full 120mL cartridges (up to 24 full arch impressions) on a single charge.

Combine PowerMix with 3rd Generation Affinity™ Impression Material

Clinically proven for outstanding impression accuracy, **Affinity Heavy Body** is designed with an ideal flow and durometer for accurate full arch tray impressions, while **Affinity InFlex**, an ultra rigid tray material is designed specifically to prevent distortion with the dual-arch technique. Both **Affinity** tray materials are available in 120mL PowerMix cartridges, which are designed to reduce plastic waste and impression material left in the cartridge. Each 120mL **PowerMix** cartridge provides enough material to fill 3 full arch trays or 6-10 quadrant impression trays. Together, impression taking has never been more efficient or more accurate.



WORKING & SET TIMES

AVAILABLE WORKING TIME

Regular Set 1:45 min
Fast Set 1:10 min

MINIMUM INTRAORAL SET TIME

Regular Set 2:30 min
Fast Set 1:45 min

2:30 min reflects the minimum intraoral set time. Our recommendation is to allow 3:00 min for a full intraoral set.

1:45 min reflects the minimum intraoral set time. Our recommendation is 2:00 min for full intraoral set.

POWERMIX AUTOMATIC DISPENSER

Contains: PowerMix Dispenser Unit, Stand, AC Adapter, Instructions

AFFINITY INFLEX REGULAR SET 120ML 2-PACK

Contains: 2 x 120mL Cartridges, 6 Dynamic Mixing Tips, Instructions/SDS

AFFINITY INFLEX FAST SET 120ML 2-PACK

Contains: 2 x 120mL Cartridges, 6 Dynamic Mixing Tips, Instructions/SDS

AFFINITY HEAVY BODY REGULAR SET 120ML 2-PACK

Contains: 2 x 120mL Cartridges, 6 Dynamic Mixing Tips, Instructions/SDS

AFFINITY HEAVY BODY FAST SET 120ML 2-PACK

Contains: 2 x 120mL Cartridges, 6 Dynamic Mixing Tips, Instructions/SDS

DYNAMIC MIXING TIP FOR POWERMIX

Contains: 50 x Dynamic Mixing Tips



Clinician's Choice

Designed for better dentistry.

clinicianschoice.com