

# Template®

## Ultra Quick Matrix Material



Traditionally, dentists use putty, alginate, vacuum stint or bite registration as a matrix to fabricate their provisional restorations. Without fail, all of these materials either take too long, lack detail or do not provide enough rigidity to prevent excess flash.

### Clinical Challenges

**Putty** – Putty takes too long to set and lacks interproximal detail resulting in excess finishing time.

**Alginate** – Requires clean up and too often produces thick margins and interproximal tags that fold during reinsertion causing excess interproximal flash and time wasted finishing your provisional restoration.

**Vacuum Form** – The most time-consuming technique with the least accuracy. Provisionals made this way lack detailed anatomy which is not worth the expensive lab time.

**Bite Registration** – This is a fast material but usually lacks detail and flow causing more finishing time. Bite registration material is often very expensive.

The best solution for your next temporary matrix is **Template® Ultra Quick Matrix Material**, a silicone matrix material that provides far superior detail and accuracy. Best of all, it sets in 30 seconds – no vacuum, no hand-mixing, and no flash! **Template** will flow into the embrasure areas and into the finest morphology, yet when cured, it will release easily and maintain its stable form indefinitely.

### Take a Template of the Lingual!

Most dentists repair Class III and IV defects only to find themselves struggling with finishing the lingual aspect of the restoration – and during the struggle the clock keeps ticking! Now, when you use **Template** and the Lingual Matrix Technique, you can pre-shape and refine the lingual surface while you're restoring the tooth, leaving only the polishing step to complete the restoration.



### The Customizable, Distortion-Free, Disposable, Metal Temporary Tray!

**TempTrays™** from **Clinician's Choice®** are designed for accuracy and versatility. Made from rigid aluminum, **TempTrays** are customizable and easy to use for both anterior and posterior temporary restorations. **TempTrays** can be easily adjusted for mal-aligned teeth and fit virtually all arch shapes. With high, perforated walls, the **TempTray** is designed to firmly retain the matrix material while eliminating the chance of distortion. **TempTrays** are the perfect companion to **Template Ultra Quick Matrix Material** and are a fraction of the cost of quadrant or dual-arch trays.

## TempTray™

- Multiple teeth can be captured
- Posterior tabs for easy placement. Can be folded in to prevent retromolar pad impingement
- Adjustable, perforated sidewalls
- Single use. 100% recyclable



&



## Anterior TempTray™

- Unique split arch design to fit all arch widths
- Adjustable handle for easy placement and removal
- High sidewall to capture any tooth length
- Perforated sidewall
- Single use. 100% recyclable





← Adjustable →

## PROVISIONAL CROWN CLINICAL TECHNIQUE

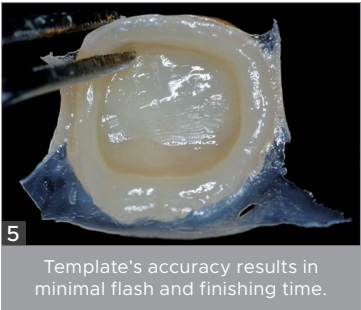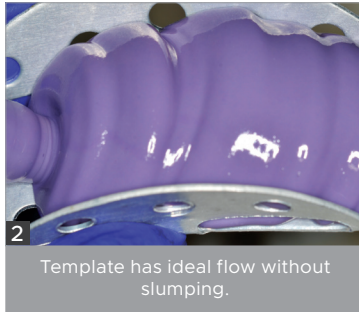

## TEMPLATE LINGUAL MATRIX CLINICAL TECHNIQUE

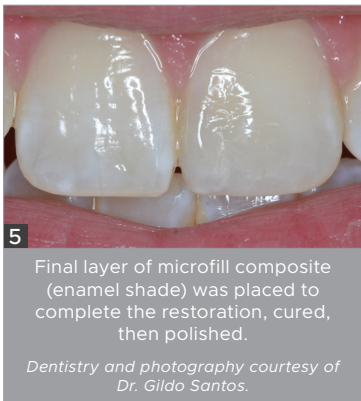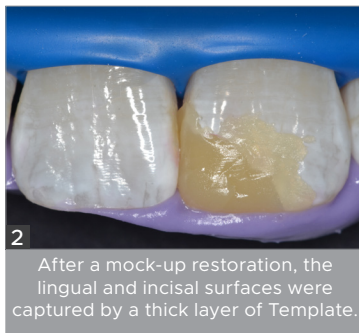

**TEMPTRAY TRIAL KIT (046152)**  
Contains: 50 x TempTrays,  
50 x Anterior TempTrays

**TEMPTRAY 50-PACK (046050)**

**TEMPTRAY 100-PACK (046100)**

**TEMPTRAY 250-PACK (046102)**

**ANTERIOR TEMPTRAY 50-PACK (046150)**

**ANTERIOR TEMPTRAY 100-PACK (046151)**

**TEMPLATE MATRIX PACK (018904)**  
Contains: 2 x 50mL Template Cartridges,  
2 x 50mL Template Clear Cartridges, Mixing Tips, Instructions/SDS

**TEMPLATE ULTRA QUICK MATRIX MATERIAL 50ML 4-PACK (028904)**  
Contains: 4 x 50mL Cartridges, Mixing Tips, Instructions/SDS

**TEMPLATE ULTRA QUICK MATRIX MATERIAL 50ML 12-PACK (018912)**  
Contains: 12 x 50mL Cartridges, Instructions/SDS

**TEMPLATE ULTRA QUICK MATRIX MATERIAL 50ML 36-PACK (018936)**  
Contains: 36 x 50mL Cartridges, Instructions/SDS

**TEMPLATE ULTRA QUICK MATRIX MATERIAL 120ML 2-PACK (021812)**  
Contains: 2 x 120mL Cartridges, 6 Dynamic Mixing Tips, Instructions/SDS

**NEW**