

Introducing

# A.S.A.P.<sup>®</sup> Dailies

## All Surface Access Diamond Polishers



### Polish for a Day and Throw Away

Take esthetics to a new level of efficiency and convenience with **A.S.A.P.<sup>®</sup> Dailies**, the next generation of daily-use polishing from **Clinician's Choice<sup>®</sup>**.

Designed for multi-use in a single day, **A.S.A.P. Dailies** flexible spiral shape provides all surface access polishing and a fast, high gloss luster on all direct and indirect restorations and all restorative materials including: composite, ceramic, zirconia, lithium disilicate and provisionals. Delivering consistent results after multiple sterilization cycles,\* diamond impregnated **A.S.A.P. Dailies** reduce your cost-per-use without sacrificing performance – simply polish for a day and throw away!

With no messy polishing paste required, the first step in the **A.S.A.P. Dailies** polishing sequence is a 15-20 second polish with the purple, 45 micron diamond impregnated Pre-polisher, followed by a light wiping motion with the peach Final High Shine Polisher (3-6 micron diamond particles) for a brilliant shine in as little as 30 seconds.

\*Minimum 3 sterilization cycles.

### CLINICAL TECHNIQUE

Dentistry and photography courtesy of Dr. Bob Margeas



**A.S.A.P. DAILIES ALL SURFACE ACCESS DIAMOND POLISHERS 50-PACK (133160)**  
Contains: 50 Polishers (25 x Pre-polishers, 25 x Final High Shine Polishers)

NEW