

Clinician's Choice Esthetic Contouring Instruments

Highly detailed anterior composite restorations require specifically designed instruments to properly manipulate and contour the material, especially along the cervical margins. The **Clinician's Choice® Esthetic Contouring Instruments** make anterior composite restorations easy!

Featuring a convex shaped paddle on one end, the Gold **Esthetic Contouring Instrument** allows you to spread composite across the facial plane for diastemas, Class IV, and Class V restorations. With other instruments, manipulation of composite is more tedious and often results in an irregular composite surface. Often, it becomes necessary to go over the composite many times in an attempt to smooth the material, when traditional instruments are used. The convex surface of the Gold **Esthetic Contouring Instrument** has no edge to catch and pull the composite, allowing for a smooth composite to tooth transition.




The convex paddle end allows for easy and smooth spreading of composite across the facial plane.

Gold Grip

Features a convex paddle to spread composite across the facial plane for diastemas, Class IV and Class V restorations. The long, flat, thin, flexible blade makes interproximal composite placement and manipulation a breeze.

Blue Grip

Bladed end allows trimming of excess composite and to cut grooves into the material to create the illusion of enamel cracks. With its opposing blade, ripples can be created prior to cutting for better light reflection.

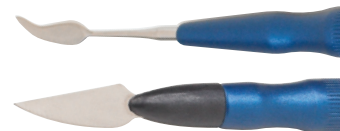

CLINICAL TECHNIQUE



The Gold Instrument's permanent end is a sturdy, curved spoon shape, the back of which is ideal for shaping the curves of the lingual concavities. The front side is used to form the curvature at gum line or interproximal on the lingual side, and may also be used to carry material to the preparation.



The replaceable (black tip) end of the Gold Instrument is spatula-shaped and has been designed for use on the facial surfaces of maxillary incisors and the buccal of posterior teeth, or where there is a large and rounded surface. Excellent for adapting a gum line margin, it is also ideal for forming the gentle curve toward the interproximal and tooth's edge for esthetic highlighting.

ESTHETIC CONTOURING INSTRUMENTS
 Gold Grip (063103)
 Blue Grip (063106)
 Gold Replacement Tip (063104)
 Blue Replacement Tip (063107)