


Esthetic Contouring Instruments

Highly detailed anterior composite restorations require specifically designed instruments to properly manipulate and contour the material, especially along the cervical margins. The Clinician's Choice® **Esthetic Contouring Instruments** make anterior composite restorations easy!

- The Gold Grip features a convex paddle to spread composite across the facial plane for diastemas, Class IV and Class V restorations. The long, flat, thin and flexible blade makes interproximal composite placement and manipulation a breeze.
- The Blue Grip features a bladed end which allows trimming of excess composite and to cut grooves in the material to create the illusion of enamel cracks. With the opposing blade, ripples can be created prior to cutting for better light reflection.

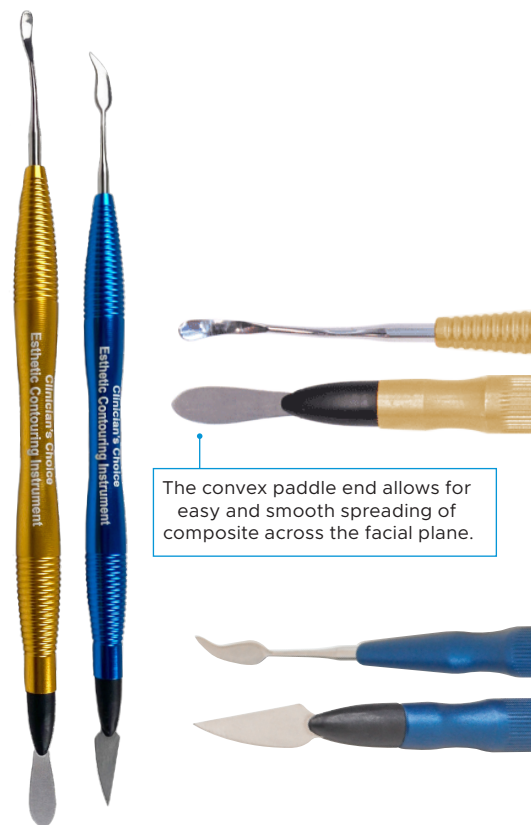

CLINICAL TECHNIQUE



The Gold Instrument's permanent end is a sturdy, curved spoon shape, the back of which is ideal for shaping the curves of the lingual concavities. The front side is used to form the curvature at gum line or interproximal on the lingual side, and may also be used to carry material to the preparation.



The replaceable (black tip) end of the Gold Instrument is spatula-shaped and has been designed for use on the facial surfaces of maxillary incisors and the buccal of posterior teeth or where there is a large and rounded surface. Excellent for adapting a gum line margin, it is also ideal for forming the gentle curve toward the interproximal and tooth's edge for esthetic highlighting.

ESTHETIC CONTOURING INSTRUMENTS

Gold Grip (063103)
Blue Grip (063106)
Gold Replacement Tip (063104)
Blue Replacement Tip (063107)

To view the latest news, education and more, visit:

www.clinicianschoice.com [f](#) [in](#) [@](#)

 **Clinician's Choice®**
Designed for better dentistry.