

Clinician's Choice
www.clinicianschoice.com

91%
overall
rating



Description

Evanescence™ Universal Restorative is a nano-enhanced, light-cured, packable and esthetic resin-based composite. It is designed to blend with the surrounding hard tissues, offers high polishability, and can be used for simple and complex restorations. It contains rheological modifiers that provide for its putty-like handling and its ability to adapt to preparations and remain in place. **Evanescence Universal Restorative** is available in enamel (E), universal (U) and dentin (D) shades with 80%, 85% and 90% opacity, respectively, and also in three non-Vita Enamel FX shades. It is delivered in 4 g syringes and pre-loaded 0.25 g single-use tips in shades: A1U, A2U, A3U, A3.5U, A4U*, B1U, B2U, C1U*, C2U, C3U*, A1E, A2E, A3E, A3.5E, B1E, B2E*, C2E+, A1D, A2D, A3D, A3.5D, B1D, C2D, C4D*, as well as bleach shades BL1* and BL3*. Specialty shades available: Enamel FX Clear+, White+, and Incisal+.

* Available in single use tips only.
+ Available in syringes only.

Indication

- All anterior and posterior direct restorations.

Unique Features

- The refractive index of its unique nano pigments and fillers are optimized through a proprietary process.
- Rheological modifiers provide for adaptability during packing and prevent slumping.

“THE BEST BLENDING COMPOSITE I HAVE EVER USED. IT BLENDS BEAUTIFULLY INTO THE TOOTH.”

Consultants' Comments

- *“Great handling. Easy to dispense, packs very nicely and stacks very well.”*
- *“I liked its putty-like consistency. It doesn't stick to instruments, pull back or slump.”*
- *“Easy to sculpt, contour, finish and polish.”*
- *“Evanescence handles with excellence. The final contour maintains its shape and blends into the natural tooth prior to light curing.”*
- *“Good shade matches and polishability. Universal shades are excellent.”*
- *“Nice polish and blends well. A good finish is achieved quickly.”*
- *“Esthetics are exceptional. Great shade match incisally and very life-like.”*
- *“I liked the opaque shades for masking dark areas and implant crown screw access holes.”*


Evaluation Highlights

Evanescence Universal Restorative was evaluated by 34 consultants, with a total of 677 uses.


- Easy dispensing and manipulation.
- Putty-like, non-sticky consistency.
- Good polishability.
- Blends with tooth structure.
- Esthetic.

Clinical Tips

- The dentin and enamel shades in combination look great in the anterior region. It's worth using both of these shades.
- Great for posterior composites.
- Use the universal shade for medium-sized restorations.
- Over-build to contour it.
- Press down with a burnisher along the margins to adapt it.
- Warm it before placement.
- The enamel shade is nice for anteriors, somewhat translucent.

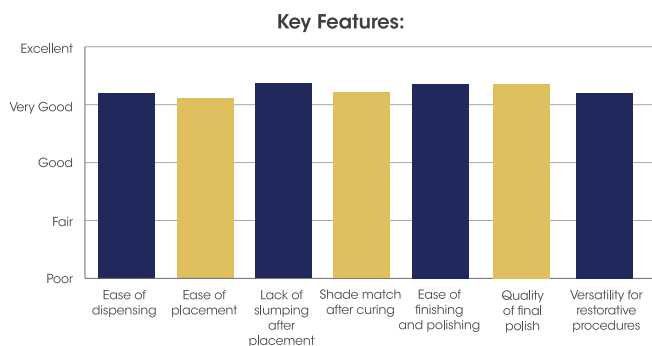

Compared to Competitive Products:

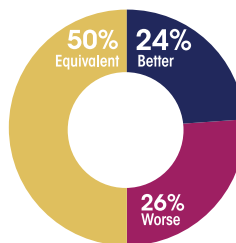

Percentage of Consultants Who Would:

