




CUSTOMIZABLE IMPRESSION TRAYS

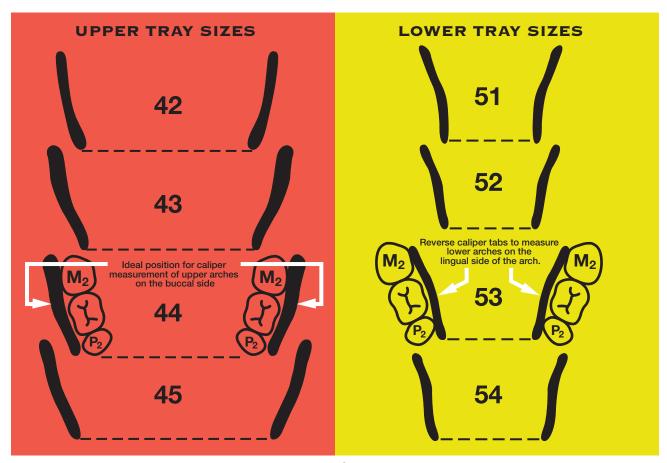

(

CLINICIAN'S CHOICE

1-800-265-3444

Phone: (519) 641-3066 • Fax: (519) 641-3083 • www.clinicianschoice.com





THE FIRST SEMI CUSTOM TRAY THAT CAN BE ADAPTED BY HAND



Mouldable HEATWAVE dentulous (upper) tray.



- Keep the tray immersed in water (70°C/158°F) for one minute.
- Use of a hot water bath is ideal
- A simple alternative is to use hot water (70°C/158°F) in a suitable container



- Adapt the tray by hand
- Keep desired form stable for 5 seconds
- Check the fit in the mouth and if needed, repeat the moulding procedure. If desired fit is achieved, run the tray under cold water for 5 seconds to set the tray's form.

NOTE: Measuring calipers can be autoclaved at 134°C/273°F for 15 minutes

CLINICIAN'S CHOICE

1-800-265-3444

Phone: (519) 641-3066 • Fax: (519) 641-3083 • www.clinicianschoice.com