Contours

Finishing & Polishing Discs

Unlock the esthetic potential of your composite by incorporating **Contours™ Finishing** and **Polishing Discs** into your polishing regimen. **Contours Finishing and Polishing Discs** are a color-coded, easy to use sequence of 4 semi-flexible Aluminum Oxide discs, designed to efficiently shape and refine your composite restoration, while polishing it to a life-like luster.



Contours Discs are available in the two most popular sizes of 3/8" and 1/2" diameters that effortlessly and securely pop onto a round mandrel. From coarse grit to superfine, the Contours Disc has a consistent flex that allows for control without gouging at the margin and minimizes any scarring of the surface from accidental contact with the metal center—no matter which direction the disc is facing. Intuitive shading of the discs, from dark purple to light pink as the grit decreases from coarse to superfine, allows for easy identification of each step of the polishing sequence. Choose the 3/8" disc for margins and areas near the papilla, as well as on posterior and/or pedodontic restorations. The larger 1/2" disc is perfect for larger surface reduction, contouring and polishing.

The **Contours Finishing and Polishing Disc** system is designed to be used in sequence from coarse to superfine; however, the coarse and medium grit discs can also be used to establish surface texture as well as preparing the composite for the A.S.A.P. Pre- and Final High Shine Polishers as part of an esthetic and efficient finishing and polishing regimen.

(n)		Coarse	Medium	Fine	Superfine
	3/8" 9.5mm	0	0	0	0
	1/2" 12.7mm	0	0	0	0

CONTOURS FINISHING & POLISHING DISCS TRIAL KIT

Contains: 200 discs (25ea: 1/2" Coarse, Medium, Fine, Superfine, 25ea: 3/8" Coarse, Medium, Fine, Superfine), 1 mandrel

CONTOURS FINISHING & POLISHING DISCS STARTER KIT (1/2")

Contains: 400 discs total (100ea: 1/2" Coarse, Medium, Fine, Superfine), 1 mandrel

CONTOURS FINISHING & POLISHING DISCS STARTER KIT (3/8")

Contains: 400 discs total (100ea: 3/8" Coarse, Medium, Fine, Superfine), 1 mandrel

CONTOURS FINISHING & POLISHING DISCS REFILLS

Contains: 85 discs 1/2" Coarse 1/2" Medium 1/2" Fine 1/2" Superfine 3/8" Coarse 3/8" Medium 3/8" Fine 3/8" Superfine

CONTOURS FINISHING & POLISHING DISCS MANDREL REFILL

Contains: 3 mandrels

