

Inspire™

Esthetic Provisional Composite



Provisional fabrication with **Inspire™** is easy, efficient, and predictable. Inspire offers superior handling for creating strong and esthetic provisionals that ensure continued tooth and tissue health, along with proper function and position.

- Inspire offers high flexural and compressive strength with an ideal elastic modulus (3136 MPa), making it easily trimmed and resistant to fracture along margins.
- Inspire is stackable, stays where you place it and offers low shrinkage (3.4%) preventing missed margins.
- Inspire shines with minimal effort, achieve a composite-like finish in seconds.
- Provides an efficient 40-second working time, can be removed from the mouth in as little as 1:30-2:30 minutes and a total cure time of 4:30 minutes to ensure procedural efficiency.
- Available in 4 shades (71g cartridges: B1, A1, A2, and A3.5) to provide ideal translucency and fluorescence to match tooth structure in all light conditions.



Dentistry and photography courtesy of
Danièle Larose, DMD, AAACD


Compare Inspire's Superior Mechanical and Physical Properties

	Elastic Modulus (MPa)	Flexural Strength (MPa)	Impact Resistance (MPa)	Compressive Strength (N/mm ²)	Diametral Tensile Strength (MPa)	Maximum Temperature (°C)	O ₂ Inhibited Layer (mg)
Inspire™	3136	105	76	379	62	53	39.4
Integrity™ *	3055	93	63	277	50	59	32.5
Integrity Multi-Cure™ *	3367	104	65	377	42	61	71.4
Luxatemp Plus™ *	2888	97	76	295	47	67	26.9
Protemp Plus™ *	2796	88	69	488	57	68	25.3
Structure 2™ *	3387	106	59	274	47	81	50
Structure 3™ *	2178	87	53	481	45	69	53.5

* Not a trademark of Clinician's Choice® Dental Products Inc. ** In-house data.

Provisional Crown Technique

Dentistry courtesy of Brad Carson, DDS. Photography courtesy of Gildo Santos, DDS.

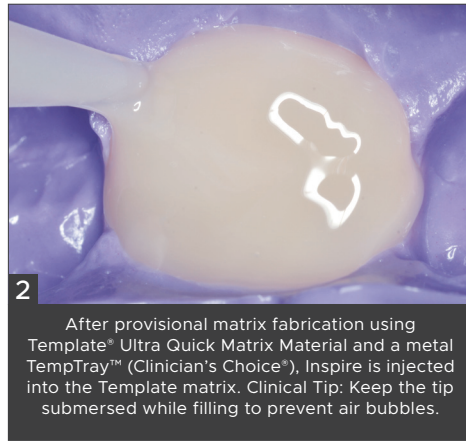

INSPIRE ESTHETIC PROVISIONAL COMPOSITE CARTRIDGES

Contains: 71g cartridge, 12 x mixing tips, instructions/SDS

- A1 (152701)
- A2 (152702)
- A3.5 (152703)
- B1 (152704)

To view the latest news, education and more, visit:

www.clinicianschoice.com   

