



# A.S.A.P.® Indirect+ All Surface Access Polishers

FINISHING & POLISHING



## Chairside Zirconia Polishing Made Easy

The A.S.A.P.® Indirect+ system of adjusters and diamond-impregnated polishers allow for chairside refinement and create an outstanding shine on all your indirect ceramic restorations.

- **All Ceramic Polishing:** Designed with a high concentration of diamond particles coupled with a durable silicone binder for use with all indirect substrates.
- **Chairside Refinement:** Multi-shaped adjusters allow for minor adjustments or the ability to remove scratches often caused by bur adjustments.
- **Fast, High Shine:** Diamond-impregnated spiral wheel polishers, Pre-polisher (blue) and Final Polisher (pink) restore lab-fabricated polish after adjustment.

## CLINICAL TECHNIQUE


### Chairside Adjustment and Polishing with the A.S.A.P. Indirect+ System



1 To make minor adjustments or to remove scratches caused by diamond bur adjustments, the A.S.A.P. Indirect+ Coarse Adjuster; available in a disc, point or cylinder shape, can be used on any zirconia, lithium disilicate or ceramic prosthesis.



3 The A.S.A.P. Indirect+ Pre-polisher is used to smooth the prosthetic surface and establish an initial gloss.

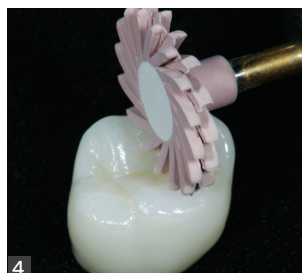

4 The A.S.A.P. Indirect+ Final Polisher creates a stunning high luster in seconds on any zirconia, lithium disilicate or ceramic surface.



Dentistry and photography courtesy of Robert Lowe, DDS



Scan to Read  
Dental Advisor Editor's Choice ++++½ Evaluation

**A.S.A.P. INDIRECT+ INTRAORAL/RA STARTER KIT (130918)**  
Contains: 3 Coarse Adjusters (1 x Point, 1 x Disc, 1 x Cylinder),  
1 x Pre-polisher, 1 x Final Polisher

**A.S.A.P. INDIRECT+ INTRAORAL/RA POLISHER REFILL 2-PACK (130919)**  
Contains: 1 x Pre-polisher, 1 x Final Polisher

**A.S.A.P. INDIRECT+ INTRAORAL/RA REFILL 3-PACKS**  
Pre-polisher (130920)  
Final Polisher (130921)  
Point Adjuster (130922)  
Cylinder Adjuster (130923)  
Disc Adjuster (130924)

**A.S.A.P. INDIRECT+ EXTRAORAL/HP STARTER KIT (130940)**  
Contains: 2 Coarse Adjusters (1 x Disc, 1 x Cylinder),  
2 x Pre-polishers, 2 x Final Polishers

**A.S.A.P. INDIRECT+ EXTRAORAL/HP POLISHER REFILL 2-PACK (130941)**  
Contains: 1 x Pre-polisher, 1 x Final Polisher

**A.S.A.P. INDIRECT+ EXTRAORAL/HP REFILL 3-PACKS**  
Pre-polisher (130942)  
Final Polisher (130943)  
Cylinder Adjuster (130944)  
Disc Adjuster (130945)



To view the latest news, education and more, visit:

[www.clinicianschoice.com](http://www.clinicianschoice.com)   

 **Clinician's Choice®**  
Designed for better dentistry.