

Optional Tack-Cure Technique

To control excess flow on the maxillary arch or in large preparations, a brief 1–3 second tack-cure can be used to prevent further composite movement. Then, by allowing the material to fully polymerize chemically for 60 seconds (self-cure), you maximize Evanesce Bulk Cure's Bio-Adaptive™ benefits for an optimal marginal seal and gap-free result.


Step 1: Tack-Cure

Tack-cure for 1-3 seconds to control the excess flow and prevent further composite movement.



Step 2: Final Cure

Allow Evanesce Bulk Cure to self-cure for 60 seconds to ensure an optimal marginal seal and gap-free result.

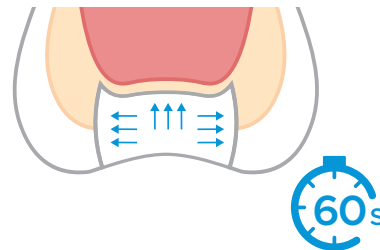

Learn more about
the Evanesce family
of restoratives.